ImuStab-1 / ImuStab-2

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English

INTENDED USE

ImuStab-1 and ImuStab-2 are stabilizer solutions for erythrocytes. They serve for the preservation of erythrocytes and the antigens/antigenic properties of which shall be kept stable for further tests (e. g. to clarify ambiguous results, antibody identification, etc.).

On storage as anticoagulated whole blood or in physiological NaCl solution the erythrocytes lose part of their potency as antigens after a few days. This results in weaker reactions towards several blood grouping systems, using homologous antisera or immune antibodies.

The solutions are intended to be used by qualified and technical personnel only.

PRINCIPLE OF PROCEDURE

ImuStab-1 and ImuStab-2 stabilizer solution are special glycine-saline solutions, which provide stability and preservation of various antigens for at least 4-8 weeks when the erythrocytes are stored in a 0.8 to 3% suspension at 2 to 8 °C.

WARNING

ImuStab-1 and ImuStab-2 contain neomycin sulfate, chloramphenicol and amphotericin B as preservatives.

These stabilizer solutions should be looked upon as potentially infectious because of never complete exclusion of danger through excitants of disease and should be handled with appropriate care.

On disposal, flush with large quantities of water.

LAGERUNG

Store opened and unopened products well closed at 2 to 8°C.

In principle, store and use the reagents to declared expiry date only.

HINWEISE

1. Inappropriate storage impairs efficacy of the reagent
2. Weak turbidity / yellowing of the reagent does not affect its reactivity.

SOLUTION PREPARATION

There is no preparation of the stabilizer solutions required.

They may be used immediately.

PROCEDURE

Not included but necessary materials:

1. test tubes (10 x 75 mm or 12 x 75 mm)
2. microliter pipette
3. centrifuge
4. isotonic saline (0,85 - 0,9% sodium chloride)

Test procedure

Conservation of Erythrocytes

1. Centrifuge fresh whole blood anticoagulated with EDTA or sodium citrate for 2 minutes at 3000 rpm (about 1500 - 2000 x g).
2. Pipette / aspirate the supernatant / plasma and buffy coat.
3. A 0.8 to 3% suspension with the desired stabilizer solution is generated from the sediment. Mix by slight panning.
4. Storage the suspension at 2-8°C

Preparation of a ready-to-use erythrocyte suspension from stabilized erythrocytes

Will the stabilized erythrocytes be needed within a 0,8 to 3 % suspension, they may be used immediately without any further preparation.

Before further use, mix the erythrocyte suspension by gently swivelling.

When requesting a different suspension intensity or dilution medium:

1. Collect a part of the suspension, transfer to a labeled test tube and centrifuge for 2 minutes at 3000 rpm (about 1500 - 2000 x g).
2. Pipette / aspirate and discard supernatant/stabilizer solution.
3. Fill the tube with isotonic saline, centrifuge again for 2 minutes at 3000 rpm (about 1500 - 2000 x g) and discard the supernatant.
4. Repeat step 3 twice (± washing the erythrocyte sediment three times).
5. From the washed sediment, create the requested test erythrocyte suspension in the desired dilution medium.
6. Store this cell suspension in a cool place and do not use it more than 12 hours

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REF	Product Code	LOT	Lot
	Store from - to		Expiration Date
IVD	In-Vitro Diagnostic		observe instruction for use
	Manufacturer as to 98/79/EG		

REF

05.203-1K	ImuStab-1	1000 ml
05.204-1K	ImuStab-2	1000 ml

730-13-5903 Version 003 / 20. January 2022

 ANTITOXIN GmbH Industriestraße 88 69245 Bammental Deutschland
 +49 (0) 6223/ 8661-0  +49 (0) 6223/ 8661-13  gara@antitoxin-gmbh.de

ImuMed