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REF
02.189-03 Anti-s for card method Coombs-reactive, polyclonal, human 3 ml
02.189-05 Anti-s for card method Coombs-reactive, polyclonal, human 5 ml

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Anti-s for card method Coombs-reactive, polyclonal, human

English

FOR IN VITRO DIAGNOSTIC USE ONLY

INTENDED USE

The reagent is used to in-vitro determine qualitative whether human red blood cells possess or lack the corresponding blood group antigen s. The reagent is intended to be used by qualified and technical personnel only to perform immunohematology screening tests as part of the practice of transfusion medicine in the general population. The test method used with this reagent is based on the principle of agglutination, performed automated as well as not automated. Normal human erythrocytes, possessing the corresponding antigen, will agglutinate in the presence of the specific antibody directed toward the antigen.

INDICATION / CONTRA-INDICATION

The blood grouping reagent is used to test patient or donor red cells for the presence of the s antigen. Typing of donor cells facilitates the selection of suitable antigen-negative units for transfusion to patients with this antibody. Cell typing also serves as final verification of the identification of Anti-s in patient or donor sera. There is no contra-indication to perform the in-vitro-test on blood samples. The product was validated with sample collected in Europe from patients of unknown ethnic background.

REAGENTS

The listed reagents are available in following formulation: Anti-s for card method Coombs-reactive, polyclonal, human The reagents contain <0.1% (w/v) sodium azide as preservative. Beside the parts active antibody and human serum, the reagents contain sodium chloride, macromolecules and bovine albumin, which has been tested and certified by the US Veterinary service inspectors.

WARNING

The reagents are prepared from human plasma. The raw materials for these reagents are tested for HBsAg, HIV 1/2- and HCV-antibodies and found to be negative. Nevertheless as biological products it should be looked upon as potentially infectious because of never complete exclusion of danger through excitants of disease. The reagents contain sodium azide that may be toxic and may react with lead or copper to form highly explosive salts. On disposal, flush with large quantities of water. For the reasons mentioned above these reagents should be handled with proper care.

STORAGE REQUIREMENT

Store opened and unopened products at +2 to +8°C. May be at room temperature while in use. In principle, store and use the reagent to declared expiry date only.

REMARKS

- With each testing positive and negative controls should be performed.
- Inappropriate storage impairs efficacy of the reagents.
- Weak turbidity of the reagents does not affect its reactivity. Bacteria and chemical contamination of the products should be avoided. If a visible change is detected, the reagent should no longer be used this sign may indicate a microbiological contamination.
- Strength of positive reactions also depends on age of used blood.
- Use the appropriate card centrifuge. The use of another card-specific centrifuge (each card centrifuge has its specified unchangeable g-force) may lead to false results due to the changed g-force.
- The test methods identified below are for manual testing only and for the devices listed in this insert. When using other automated or semi-automated instruments, follow the procedures that are contained in the operator's manual provided by the device manufacturer. Laboratories must follow approved validation procedures.
- For usage of these reagents all effective national laws, directives and guidelines have to be observed, in Germany especially the „Richtlinien zur Gewinnung von Blut und Blutbestandteilen und zur Anwendung von Blutprodukten (Hämotherapie)“ in its actual form.
- The information on the use of the test cards in the relevant insert must be observed.

SAMPLE PREPARATION

- Blood samples should be collected by approved medical procedure.
- Blood samples to be tested should be used as soon as possible after blood collection to reduce the risk of false-positive and false-negative results due to improper storage or contamination of the sample. If a delay in testing occurs, samples should be stored at +2 to +8°C. Blood drawn into EDTA should be tested within 7 days and samples treated with sodium citrate within 14 days after collection. Blood bag / Donor Blood can be tested until the expiry date.

REAGENT PREPARATION

There is no preparation of the reagents required. Take and use the reagents directly from the vials.

PROCEDURE

Not included but necessary materials:

Card Method manuall:

- Cards: - Grifols „DG Gel Coombs“ - Ortho BioVue® System AHG polyspecific or - Ortho BioVue® System Anti-Humanglobulin Anti-IgG
- Microliter pipette
- Tubes
- Isotonic saline (0,85 - 0,9% sodium chloride)
- Centrifuge
- Card Centrifuge
- Card Incubator
- Card specific diluent

Card Method automatic:

Vision® Analyzer

Test procedure

Card Method (manual method / valid for the card: - Grifols „DG Gel Coombs“)

- Prepare 0,8 % suspensions of red blood cells in card specific diluent (red blood cells may be washed 1-3 times with isotonic saline).
- Add 50 µL of appropriate cell suspension to each marked micro tube.
- Add 25 µL of the appropriate reagent to each micro tube.
- Incubate card at +37°C for 15 minutes in a Grifols card incubator.
- Centrifuge the card in a Grifols card centrifuge with the for this centrifuge unchangeable g-force.
- Check macroscopically for agglutination within 30 minutes.
- Document the result.

Ortho BioVue® System (manual method)

- Prepare 3-5 % suspensions of red blood cells in card specific diluent (red blood cells may be washed 1-3 times with isotonic saline).
- Add 40 µL of the appropriate reagent to the appropriate marked reaction chambers.
- Add 10 µL of appropriate cell suspension to each reaction chamber.
- Incubate card at +37°C for 15 minutes in an adequate incubator.
- Centrifuge the cassette in appropriate centrifuge with the for this centrifuge unchangeable g-force.
- The test results should be read directly after the end of centrifugation.
- Document the result.

Ortho BioVue® System (Ortho Vision® Analyzer)

The use of the mentioned above reagents are validated on the Ortho Vision® Analyzer The Ortho Vision® Analyzer offers the following options / settings: - 50 µL 0,8% suspension of red blood cells in PBS buffer or - 10 µL 3-5% suspension of red blood cells in PBS buffer

INTERPRETATION OF RESULTS

- Positive results (+): Visible agglutination of erythrocytes is a positive result and indicates the presence of the corresponding antigen.
- Negative results (-): No visible agglutination of erythrocytes is a negative result and indicates the absence of the corresponding antigen.

Read and interpret the results of the card method according to the instruction for use of the corresponding card. The interpretation of the results from Ortho Vision® Analyzer is performed automatically by the software.

LIMITATIONS OF THE PROCEDURE

- Inaccuracy at compliance with instructions written under section "Procedures" and "Interpretation of results" may lead to incorrect results.
- No valid conclusion concerning the test result can be reached, if controls with uncertain or false results occur.
- Enzyme treated erythrocytes or addition of bovine albumin and/or other solutions containing protein may cause unspecific reactions.
- Hemolyzed, turbid, contaminated or clotted blood samples must not be used in this test.
- Due to variability of antigen expression on human red blood cells, reactivity of the reagents mentioned above, against certain phenotypes, may give weaker reactivity compared to control cells.
- No one specific antiserum or technique can be guaranteed to detect all rare, weak or variants of antigens.²
- Red blood cells coated with alloantibodies or autoantibodies of the same or similar specificity, as that reagent used for the test (i.e. cells that are positive in the direct Antiglobulin-test (DAT)) are not suitable for this test procedure.
- Red blood cells with a positive direct Coombs-test may cause false-positive reactions.
- It is described in the literature, that samples from patients treated with anti-CD38 monoclonal antibodies can cause false positive results in the Coombs test.⁵
- Information on limits in the instructions for use of cards used must be observed.
- When used on the Ortho Vision® Analyzer, the test serum may remain uninterrupted on the device (+15 to +30 °C) for a maximum of 7 days. On the other hand if the test serum is stored overnight at +2 to +8 °C for 12 hours, the period will be extended accordingly to 14 days.
- In the Grifols "DG Gel Coombs" card the test serum Anti-s for card method can in rare cases cause weak positive reactions with s-negative cells. In case of weak positive reactions, it should be confirmed with further tests.

INCIDENTS RELATED TO THE DEVICE

Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is established.

PERFORMANCE CHARACTERISTICS

A performance evaluation for the products was carried out. The required samples were used and compared with other reference methods / products.

Technique \ Product	Performance Characteristics			
	Positive Bleeds n	Sensitivity	Negative Bleeds n	Specificity
Grifols „DG Gel Coombs“	97/97	100 %	19/19	100 %
Ortho BioVue® AHG	97/97	100 %	19/19	100 %

Diagnostic Sensitivity: The probability that the device gives a positive result in the presence of the target marker.

Diagnostic Specificity: The probability that the device gives a negative result in the absence of the target marker.

The product is equivalent and does not differ in quality from comparable reagents available on the market.

DIFFERENCES BETWEEN BATCHES

Validation between three batches over the entire shelf life showed no differences.

INTERFERENCE STUDY

The interference studies showed no impairment for the qualitative test when using the following interfering substances: Heparin 720 U/dl, Albumin 15000 mg/dl, Triglycerides 1500 mg/dl, Bilirubin 40mg/dl, Ethanol 620 mg/dl, Glucose 1000 mg/dl. For the anticoagulants (EDTA, Sodium Citrate, ACD, CPD-A, PAGGS-M) three times the recommended concentration was tested.

SUMMARY OF SAFETY AND PERFORMANCE

The Summary of Safety and Performance of this reagent is available via ANTITOXIN (www.antitoxin-qmbh.de) and can be accessed via the EUDAMED database.

LITERATURE

- Richtlinien zur Gewinnung von Blut und Blutbestandteilen und zur Anwendung von Blutprodukten (Hämotherapie)
- CLSI, I/LA33-A Validatin of Automated System for Immunhematological Testing Before Implementation; Approved Guideline CLINICAL AND LABORATORY STANDARDS INSTITUTE December 2009
- Peter D. Issit, David J. Anstee Applied Blood Group Serology, 4. edition, Montgomery Scientific Publications 1998
- Christian Mueller-Eckhardt, Volker Kiefel Transfusionsmedizin, 3.edition, Springer-Verlag 2004.
- Blood Transfusion Management for Patients Treated with Anti-CD38 monoclonal Antibodies. Frontiers in Immunology November 2018 / Articiel 2616

REF Product Code	LOT Batch
Store from - to	Expiration Date
IVD In-Vitro Diagnostic	EU CE symbol
Manufacturer according to 98/79/EG	Consult instructions for use

REF

02.189-03 Anti-s for card method Coombs-reactive, polyclonal, human 3 ml
02.189-05 Anti-s for card method Coombs-reactive, polyclonal, human 5 ml

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ANTITOXIN GmbH Industriestrasse 88 69245 Bammental Alemanha