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REF

700290	Anti-Kp ^a Coombs-reactive, polyclonal, human	2 ml
700300	Anti-Kp ^b Coombs-reactive, polyclonal, human	2 ml

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Anti-Kp^a Coombs-reactive, polyclonal, human**Anti-Kp^b Coombs-reactive, polyclonal, human**

For indirect Coombs-Test

FOR IN VITRO DIAGNOSTIC USE ONLY**INTENDED USE**

Polyclonal Coombs-reactive Anti-Kp^a, Anti-Kp^b reagents are produced from human plasma that contains antibodies of IgG-type which reacts specific with the corresponding blood group antigen. The reagents are used for In-Vitro-Diagnostic, to determine qualitative, whether human red blood cells possess or lack the corresponding blood group antigen Kp^a, Kp^b. The reagents are intended to be used by qualified and technical personnel only.

PRINCIPLE OF PROCEDURE

The test method use with these reagents are based on the principle of agglutination. Normal human erythrocytes, possessing one of the appropriate antigens, will be recognizes and coated by the corresponding specific antibody and then the cells will be agglutinated by a secondary antibody that reacts with human IgG-molecules.

REAGENTS

The reagents are available in following formulation:

Anti-Kp^a Coombs-reactive, polyclonal, humanAnti-Kp^b Coombs-reactive, polyclonal, human

The reagents contain <0.1% (w/v) sodium azide as preservative. Beside the parts active antibody and human serum, the reagents contain sodium chloride, macromolecules and bovine albumin, which has been tested and certified by the US Veterinary service inspectors.

WARNING

The reagents are prepared from human plasma.

The raw materials for these reagents have been tested for HBsAg, HIV1/2- and HCV-antibodies and found to be negative. Nevertheless, as biological products they should be looked upon as potentially infectious because of never complete exclusion of danger through excitants of disease. The reagents contain sodium azide that may be toxic and may react with lead or copper to form highly explosive salts.

On disposal, flush with large quantities of water.

For the reasons mentioned above, reagents should be handled with proper care.

STORAGE REQUIREMENT

Store opened and unopened products at +2 to +8°C.

May be at room temperature while in use.

In principle, store and use the reagents to indicated expiry date only.

REMARKS

1. With each testing positive and negative controls should be performed.
2. Inappropriate storage impairs efficacy of the reagents.
3. Weak turbidity of the reagents does not affect its reactivity. Bacteria and chemical contamination of the products should be avoided. If a visible change is detected, it may indicate a microbiological contamination, the reagent should no longer be used.
4. Strength of positive reactions also depends on age of used blood
5. Centrifugation outside the specified speed range may lead to false results.
6. The test method identified below is for manual testing only. When using automated or semi-automated instruments, follow the procedures that are contained in the operator's manual provided by the device manufacturer. Laboratories must follow approved validation procedures.
7. For usage of these reagents all effective national laws, directives and guidelines have to be observed, in Germany especially the „Richtlinien zur Gewinnung von Blut und Blutbestandteilen und zur Anwendung von Blutprodukten (Hämotherapie)“¹ in its current version.

REAGENT PREPARATION

1. Blood sample should be collected by approved medical procedure.
2. Blood sample to be tested should be used as soon as possible after blood collection to reduce the risk of false-positive and false-negative results due to improper storage or contamination of the sample.
If a delay in testing occurs, sample should be stored at +2 to +8°C.
Blood drawn into EDTA should be tested within 7 days and sample treated with sodium citrate within 14 days after collection.
Blood bag / Donor Blood can be tested until the expiry date.

REAGENT PREPARATION

There is no preparation of the reagents required.

Take and use reagents directly from the vials.

PROCEDURE

Not provided material, additionally needed

- Tube Centrifugation Method:
1. tubes (10 x 75 mm or 12 x 75 mm)
 2. microliter pipette
 3. timer
 4. incubator
 5. centrifuge
 6. isotonic saline (0,85 - 0,9% sodium chloride)
 7. Anti-Human-Globulin-Reagent (Coombs-Serum / AHG- Serum)

Test procedure**Tube Centrifugation Method**

1. Prepare 2% to 5% suspensions of red blood cells in isotonic saline (red blood cells may be washed 1-3 times with isotonic saline).
2. At first put 100 µL of appropriate reagent in a marked tube, subsequently add 100 µL of appropriate cell suspension.
Alternative one drop = approximately 50 µL cell suspension can be added to one drop = approximately 50 µL reagent.
3. Mix Erythrocytes- / Reagent-mixture well by slightly shaking.
4. Incubate tube in an incubator at +37 °C for 30 min.
5. Wash red cells 3 times with (cold) isotonic saline.
6. Subsequently add 100 µL Anti-Human-Globulin-Reagent (Coombs-Serum / AHG-Serum) to the tube, release the cells from the bottom of the tube by slightly shaking and mix with the Coombs-Serum / AHG-Serum.
7. Centrifuge tube for 1 min at 1.000 rpm (approximately 180-270 x g).
8. Gently shake the red cells from the bottom of the tube and check macroscopically for agglutination within 3 minutes.
9. Document the result.

INTERPRETATION OF RESULTS

"Slightly shake up" of the erythrocyte sediment:

- Positive results (+): visible agglutination of erythrocytes is a positive result and indicates the presence of the corresponding antigen.
Negative results (-): No visible agglutination of erythrocytes is a negative result and indicates the absence of the corresponding antigen.

LIMITATIONS OF THE PROCEDURE

1. Inaccuracy at compliance with instructions written under section "Procedures" and "Interpretation of results" may lead to incorrect results.
2. No valid conclusion concerning the test result can be reached, if controls with uncertain or false results occur.
3. Enzyme treated erythrocytes or addition of bovine albumin and / or other solutions containing protein may cause unspecific reactions.
4. Hemolyzed, turbid, contaminated or clotted samples should not be used.
5. Due to variability of antigen expression, reactivity of these reagents against certain phenotypes may give weaker reactivity compared to control cells.
6. No one specific antiserum or technique can be guaranteed to detect all rare, weak or variant Antigens.²
7. Red blood cells coated with alloantibodies or autoantibodies of the same or similar specificity as that reagent used for the test (i.e., cells that are positive in the direct antiglobulin-test (DAT)) are unsuitable.
8. Red blood cells with a positive direct Coombs-test may cause false-positive reactions. The cells react positively even without a test serum.
9. As described in the literature, samples from patients treated with anti-CD38 monoclonal antibodies can cause false positive results in the Coombs test.⁵

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 REF	Product Code	 LOT	Lot
	Store from - to		Expiration Date
 IVD	In-Vitro Diagnostic	 CE	EU CE symbol
	Manufacturer as to 98/79/EG		Observe instruction for use
 UDI	Unique Device Identification		Distributor

 REF		
700290	Anti-Kp ^a Coombs-reactive, polyclonal, human	2 ml
700300	Anti-Kp ^b Coombs-reactive, polyclonal, human	2 ml

730-13-9013 Version 013a __ / 24.11.2025

 LOXO GmbH **Dieselstraße 2** D-69221 Dossenheim Germany

 ANTITOXIN GmbH Industriestraße 88 69245 Bammental Germany

 +49 (0) 6223/ 8661-0  +49 (0) 6223/ 8661-13  gara@antitoxin-gmbh.de

Marking of changes

Changes compared to the previous version are highlighted in grey