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REF

- 04.008-02** Anti-A_{tp} Lectin (Helix pomatia) 2 ml
- 04.008-03** Anti-A_{tp} Lectin (Helix pomatia) 3 ml
- 04.008-05** Anti-A_{tp} Lectin (Helix pomatia) 5 ml

Version R002 / 16.08.2024

CE 0483

 ANTITOXIN GmbH Industriestraße 88 69245 Bammental Deutschland
 +49 (0) 6223/ 8661-0  +49 (0) 6223/ 8661-13  gara@antitoxin-gmbh.de

Anti-A_{HP} Lectin (Helix pomatia)

English

for Slide- and Tube Method

FOR IN VITRO DIAGNOSTIC USE ONLY

INTENDED USE

The reagent is used to in-vitro determine qualitative as an aid in subtyping whether human red blood cells possess or lack the corresponding blood group antigens A1 and A2. The reagent is intended to be used by qualified and technical personnel only to perform immunohematology screening tests as part of the practice of transfusion medicine in the general population. The test method used with this reagent is based on the principle of agglutination, performed manually. Normal human erythrocytes, possessing the corresponding antigens, will agglutinate in the presence of the specific antibody directed toward the antigens. The reagent is used as an aid in subtyping of blood group A.

INDIKATION / CONTRA-INDIKATION

The blood grouping reagent is used to test patient or donor red cells for the presence of the A1 and A2 antigen. Typing of donor cells facilitates the selection of suitable antigen-negative units for transfusion to patients with this antibody. Cell typing also serves as final verification of the identification of Anti-A1 and A2 in patient or donor sera. There is no contra-indication to perform the in-vitro-test on blood samples. The product was validated with sample collected in Europe from patients of unknown ethnic background.

REAGENTS

The listed reagent is offered in the following form:

Anti-A_{HP} Lectin (Helix pomatia)

The reagent contains <0.1% (w/v) sodium azide as preservative. Beside the part of active lectin extract, the reagent contains sodium chloride, macromolecules and bovine albumin, which has been tested and certified by the US Veterinary service inspectors.

WARNING

This reagent was made from the animal extract of the protein gland of the vineyard snail (Helix pomatia). As biological product it should be looked upon as potentially infectious because of never complete exclusion of danger through excitants of disease. The reagent contains sodium azide that may be toxic and may react with lead or copper to form highly explosive salts.

On disposal, flush with large quantities of water.

For the reasons mentioned above the reagent should be handled with proper care.

STORAGE REQUIREMENT

Store opened and unopened products at +2 to +8°C.

May be at room temperature while in use.

In principle, store and use the reagent to declared expiry date only.

REMARKS

1. With each testing positive and negative controls should be performed.
2. Inappropriate storage impairs efficacy of the reagent
3. Weak turbidity of the reagent does not affect its reactivity. Bacteria and chemical contamination of the reagent should be avoided. If a visible change is detected, the reagent should no longer be used, this sign may indicate a microbiological contamination.
4. Strength of positive reactions also depends on age of used blood.
5. Centrifugation highly different from appointed relative centrifugal force may lead to false results.
6. The test methods identified below are for manual testing only. When using automated or semi-automated instruments, follow the procedures that are contained in the operator's manual provided by the device manufacturer. Laboratories must follow approved validation procedures.
7. For usage of this reagent all effective national laws, directives and guidelines have to be observed in its current valid version, in Germany especially the „Richtlinien zur Gewinnung von Blut und Blutbestandteilen und zur Anwendung von Blutprodukten (Hämotherapie)“¹.

SAMPLE PREPARATION

1. Blood sample should be collected by approved medical procedure.
2. Blood samples to be tested should be used as soon as possible after blood collection to reduce the risk of false-positive and false-negative results due to improper storage or contamination of the reagent. If a delay in testing occurs, samples should be stored at +2 to +8°C. Blood drawn into EDTA should be tested within 7 days and samples treated with sodium citrate within 14 days after collection. Blood bag / Donor Blood can be tested until the expiry date.

REAGENT PREPARATION

There is no preparation of the reagent required.

Take and use the reagent directly from the vials.

PROCEDURE

Not included but necessary materials:

Slide Method:

1. glass slide
2. Pasteur pipette
3. mixing stick
4. timer

Tube Centrifugation Method:

1. test tubes (10 x 75 mm or 12 x 75 mm)
2. microliter pipette
3. centrifuge
4. timer
5. isotonic saline (0,85 - 0,9% sodium chloride)

Test procedure

Slide Method

1. Use erythrocyte sediment only.
2. Place one drop (approximately 50 µL) of the reagent on a marked glass slide.
3. Using a Pasteur pipette add one drop erythrocyte sediment (approximately 50 µL) to the drop of reagent.
4. Mix the erythrocytes with reagent well with a stick and spread to a circle with a diameter of approximately 2 cm.
5. By slightly rotating the slide, check for agglutination within 1 minute (reaction starts within seconds).
6. Document the result.

Tube Centrifugation Method

1. Prepare 2% to 5% suspensions of red blood cells in isotonic saline (red blood cells may be washed 1-3 times with isotonic saline).
2. At first put 100 µl of the reagent in a marked tube, subsequently add 100 µl of appropriate cell suspension. Alternative one drop = 50 µl cell suspension can be added to one drop = 50 µl reagent.
3. Mix Erythrocytes- / Reagent mixture well by slightly shaking.
4. Incubate tube at room temperature for 5 min.
5. Centrifugation of tube for 1 minute at 1.000 rpm (approximately 180-270 x g).
6. Gently shake the red cells completely from the bottom of the tube and check macroscopically for agglutination within 3 minutes.
7. Document the result.

INTERPRETATION OF RESULTS

"Slightly rotating" at Slide Method

"Slightly shaking" at Tube Centrifugation Method:

Positive results (+): Visible agglutination of erythrocytes is a positive result and indicates the presence of the corresponding antigen.

Negative results (-): No visible agglutination of erythrocytes is a negative result and indicates the absence of the corresponding antigen.

LIMITATIONS OF THE PROCEDURE

1. Inaccuracy at compliance with instructions written under section "Procedures" and "Interpretation of results" may lead to incorrect results.
2. No valid conclusion concerning the test result can be reached, if controls with uncertain or false results occur.
3. The subtypes A1 and A2 are specifically identified with this reagent. In order to define further subgroups of blood Type A, supplementary tests are necessary.
4. Enzyme treated erythrocytes or addition of bovine albumin and/or other solutions containing protein may cause unspecific reactions.
5. Hemolyzed, turbid, contaminated or clotted blood samples must not be used in this test.
6. With the slide method, unspecific reactions might appear due to drying of the reaction-formation or if the slide is heated.
7. No one specific antiserum or technique can be guaranteed to detect all rare, weak or variant of antigens.²
8. Due to variability of antigen expression on human red blood cells, reactivity of the reagent, mentioned above, against certain phenotypes may give weaker reactivity compared to control cells.
9. Red blood cells coated with alloantibodies or autoantibodies of the same or similar specificity as that reagent used for the test (i.e., cells that are positive in the direct antiglobulin-test (DAT)) are unsuitable.

INCIDENTS RELATED TO THE DEVICE

Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is established.

PERFORMANCE CHARACTERISTICS

A performance evaluation for the products was carried out.

The required samples were used and compared with other reference methods / products.

Technique	Product	Performance Characteristics			
		Positive Bleeds n	Sensitivity	Negative Bleeds n	Specificity
Tube methode		60/60	100 %	76/76	100 %
Slide Method		60/60	100 %	76/76	100 %

Diagnostic Sensitivity: The probability that the device gives a positive result in the presence of the target marker.

Diagnostic Specificity: The probability that the device gives a negative result in the absence of the target marker.

The product is equivalent and does not differ in quality from comparable reagents available on the market.

DIFFERENCES BETWEEN BATCHES

Validation between three batches over the entire shelf life showed no differences.

INTERFERENCE STUDY

The interference studies showed no impairment for the qualitative test when using the following interfering substances:

Heparin 720 U/dl, Albumin 15000 mg/dl, Triglycerides 1500 mg/dl, Bilirubin 40mg/dl, Ethanol 620 mg/dl, Glucose 1000 mg/dl.

For the anticoagulants (EDTA, Sodium Citrate, ACD, CPD-A, PAGGS-M) three times the recommended concentration was tested.

SUMMARY OF SAFETY AND PERFORMANCE

The Summary of Safety and Performance of this reagent is available via ANTITOXIN (www.antitoxin-gmbh.de) and can be accessed via the EUDAMED database.

LITERATURE

1. Richtlinien zur Gewinnung von Blut und Blutbestandteilen und zur Anwendung von Blutprodukten (Hämotherapie)
2. CLSI, I/LA33-A Validation of Automated System for Immunohematological Testing Before Implementation; Approved Guideline CLINICAL AND LABORATORY STANDARDS INSTITUTE December 2009
3. Peter D. Issit, David J. Anstee Applied Blood Group Serology, 4. Edition, Montgomery Scientific Publications 1998
4. Christian Mueller-Eckhardt, Volker Kiefel Transfusionsmedizin, 3. edition, Springer-Verlag 2004.

REF	Product Code	LOT	Lot
	Store from - to		Expiration Date
IVD	In-Vitro Diagnostic		EU CE symbol
	Manufacturer as to 98/79/EG		Observe instruction for use
UDI	Unique Device Identification		Distributor

REF

- 04.008-02** Anti-A_{tp} Lectin (Helix pomatia) 2 ml
- 04.008-03** Anti-A_{tp} Lectin (Helix pomatia) 3 ml
- 04.008-05** Anti-A_{tp} Lectin (Helix pomatia) 5 ml

Version R002 / 16.08.2024

CE 0483

 ANTITOXIN GmbH Industriestraße 88 69245 Bammental Germany
 +49 (0) 6223/ 8661-0  +49 (0) 6223/ 8661-13  gara@antitoxin-gmbh.de